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LIST OF CONTENTS

Items Supplied	9
At A Glance	9
Symbols	10
Signal Words	10
Intended Use	11
When May the Device Not Be Used?	11
Safety Notices	12
Inserting Batteries	13
Use	13
Cleaning and Storage	15
Troubleshooting	15
Disposal	16
Technical Data	16

ITEMS SUPPLIED

- 1 x basic device
- 1 x electrode cable
- 2 x EMS pad
- 1 x operating instructions

Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the device but contact our customer service department.

Remove any possible films, stickers or transport protection from the device.

Never remove the rating plate and any possible warnings!

AT A GLANCE

(Picture A + B)

- 1 Socket for the electrode cable
- 2 Electrode cable
- 3 Pole of the electrode cable
- 4 EMS pad
- 5 Display
- 6 ON/OFF slide switch: Device on/off
- 7 Battery compartment
- 8 Metal plate
- 9 HIGH button: increase intensity
- 10 LOW button: decrease intensity
- 11 MODE button: set the EMS mode
- 12 ON button: EMS on/off

Two batteries (1.5 V; AAA) are also required.

Dear Customer,

We are delighted that you have chosen the **MAXXMEE EMS neck massage device**. Prolonged sitting at a desk, constantly looking down at a smartphone or an uncomfortable position lying down – this massage device is your discreet and yet stylish companion to alleviate these situations. Simply place it around your neck – the light headphones design ensures it fits perfectly – choose between the acupuncture, acupressure, kneading, tapping or pressure massage and the automatic programme sequence, and relax. In addition, you can adapt the intensity to suit your preference in 10 levels. The stimulation goes deep under the skin and delivers soothing pulses to the neck. With the two additional EMS pads, it can also be applied for example to the lower back, the shoulders, thighs etc.

If you have any questions about the device and about spare parts/accessories, contact the customer service department via our website: www.dspro.de/kundenservice

We hope you have a lot of fun with your **MAXXMEE EMS neck massage device**.

Information About the Operating Instructions

Before using the device for the first time, please read through these operating instructions carefully and keep them for future reference and other users. They form an integral part of the device. The manufacturer and importer do not accept any liability if the information in these operating instructions is not complied with.

SYMBOLS

Danger symbols: These symbols indicate possible dangers. Read the associated safety notices carefully and follow them.

Supplementary information

Read operating instructions before use!

Symbol for direct current

Protection class III

SIGNAL WORDS

Signal words indicate hazards if the associated notices are not followed.

DANGER – high risk, results in serious injury or death

WARNING – medium risk, may result in serious injury or death

CAUTION – low risk, may result in minor or moderate injury

NOTICE – may result in risk of damage to material

INTENDED USE

- This device is intended to be used for stimulating the skin and the tissue beneath and for electrical muscle stimulation.
 - The device is not designed for medical or therapeutic purposes. It is not a substitute for medical treatment. If complaints persist, consult a doctor.
 - The device is for personal use only and is not intended for commercial applications.
 - Use the device only for the specified purpose and as described in the operating instructions. Any other use is deemed to be improper.
 - The warranty does not cover faults caused by incorrect handling, damage or attempted repairs. The same applies to normal wear and tear.
- women during their period,
 - people who suffer from circulation problems or thrombosis,
 - people who have undergone an operation for which the treatment is not yet complete,
 - people with physical abnormalities,
 - people with a malignant tumour,
 - people with nerve damage,
 - people under the influence of medication,
 - people who belong to a risk group or who have concerns about their health.
- Do not use the device in the vicinity of or on:
 - regions which have recently been operated on or are scarred,
 - injuries and areas of swelling,
 - acne, birthmarks and warts,
 - injured, inflamed, (sun-)burnt, irritated or very sensitive skin,
 - bones and joints,
 - the rib cage close to your heart,
 - the genital area,
 - body parts with no feeling.

WHEN MAY THE DEVICE NOT BE USED?

- Children, people who are sleeping or unconscious and animals must not be treated with the device.
 - Do not use the device if you have a fever or illness/dental problems.
 - Do not use the device while you are connected to medical equipment, e.g. an ECG machine.
 - Do not use the device if you have calf pain.
 - The following people may use the device **only following consultation with a doctor**:
 - people who are fitted with a pacemaker or similar medical implants,
 - people with heart disease or who suffer from epilepsy,
 - people who have spinal disorders or back complaints,
 - women who are pregnant or have just given birth,
- If you are unsure whether the device may be used, consult a responsible doctor.

SAFETY NOTICES

- This device is not intended to be used by people (including children) with reduced sensory or mental abilities or a lack of experience and/or knowledge, unless they are supervised by a person who is responsible for their safety or have received instruction from this person on how to use the device safely. There is a danger of injury from incorrect use!
- Do not make any modifications to the device. It may only be taken apart and / or repaired by the manufacturer, customer service department or a similarly qualified person (e.g. specialist workshop) in order to avoid any hazards.

WARNING: Do not use this device near a bathtub, shower or any vessel containing water. Never use while bathing or showering!

WARNING – Danger from Batteries

- Batteries can be deadly if swallowed. Batteries and the device should therefore be kept out of the reach of children and animals. If a battery is swallowed, seek medical assistance immediately. There is a danger of burns from battery acid.
- If a battery leaks, do not allow your skin, eyes and mucous membranes to come into contact with the battery acid. Wear protective gloves if necessary. If you touch battery acid, rinse the affected areas with abundant clear water straight away and seek medical assistance immediately. There is a danger of burns from battery acid.
- Use only the battery type specified in the technical data.
- Always replace all batteries together. Do not use batteries of different types

or brands, or batteries with different capacities.

- Remove the batteries when they are used up or the device is not going to be used for a prolonged period. This will prevent any damage caused by leaks.
- Remove batteries that have leaked immediately and clean the contacts in the device.

WARNING – Danger of Fire

- Keep the device away from combustible materials and substances.

WARNING – Danger of Injury

- Keep children and animals away from the packaging material. There is a danger of suffocation.
- Make sure that the electrode cable is always kept out of the reach of small children and animals. There is a danger of strangulation.
- Children must be supervised to ensure that they do not play with the device. There is a danger of injury from incorrect use.
- Do not use the device while driving a vehicle.
- Do not use the device if you are attached to high-frequency surgical devices or are in the vicinity of such devices or high-frequency commercial equipment.
- Do not use the device in the immediate vicinity of devices which operate using short waves or microwaves. Maintain a minimum gap of 1 metre.
- The device must not be used near a mobile phone. Do not use a mobile phone while using the device.
- If you experience an unpleasant feeling while using the device, stop using it immediately!

- Stop using the device if you experience skin redness or discolouration that lasts for more than 24 hours.
- Do not use the device if it has malfunctioned, fallen into water or if there is visible damage on the device or accessories. Have it checked in a specialist workshop before using it again.

NOTICE – Risk of Damage to Material and Property

- Use and store the device only in closed rooms.
- Do not operate or store the device in rooms with high humidity.
- Never immerse the device in water or other liquids and ensure that these components cannot fall into water or become wet.
- Do not cover the device during operation.
- Use and store the device at room temperature. Do not expose the device to any extreme temperatures, temperature fluctuations, fire, heat sources and prolonged direct sunlight.
- Protect the device from impacts.
- Do not place heavy objects on the device or the accessories.
- In the event of damage / faults, switch the device off immediately.
- Use only original accessories from the manufacturer in order to guarantee that there is no interference that may prevent the device from working and to avoid possible damage.

INSERTING BATTERIES

Before the device is first used, two 1.5V batteries AAA (not supplied with the device) must be inserted.

Before inserting the batteries, check whether the contacts in the device and on the batteries are clean, and clean them if necessary.

1. Open the battery compartment (7):
Use a finger nail to reach under the catch of the battery compartment cover and remove it.
2. Insert two 1.5V batteries AAA into the battery compartment. Pay attention to the polarity (+ / -).
3. Reinsert the battery compartment cover and flip it shut.

Replace the batteries when the power of the device starts to wane.

USE

EMS stands for electrical muscle stimulation. The basic device is used to stimulate the neck muscles. In addition, you can use the EMS pads (4) to treat the muscles of the abdomen, legs, arms, back or buttocks.

Beginners should treat one muscle group for no more than 15 minutes a day to prevent overstimulation!

Advanced users can use the device for up to 30 minutes a day on each muscle group. We recommend no more than 2 units per day, with a rest phase of at least 4 hours between the units.

Please Note!

- During the treatment, there must always be moisture between the skin and the metal plates.
- Do not use the device on your face, head or the area of the thyroid glands.
- Do not use straight after eating.

- The device should not be used before going to sleep. Massages have a stimulating effect, which may lead to problems getting to sleep.
- For reasons of hygiene, the device should only be used by one person and cleaned after each use (see the "Cleaning and Storage" chapter). Keep the device clean.
- The skin may feel warm during the treatment. This is normal and not a cause for concern.
- Do not bend the EMS pads.
- The EMS pads and the metal plates must not come into contact during use with metal parts, e.g. on belts, wrist watches, jewellery or similar items.
- We explicitly point out that with some people skin irritations or hypersensitivity may occur as a result of the electrical muscle stimulation. In this case, stop using the device immediately and consult a doctor if necessary.

Neck Massage

1. Wipe the neck with a damp cloth.
2. Wet the two metal plates (8) with water.
3. Hang the neck massage device around your neck so that the metal plates are resting against your neck (Picture C).

The EMS pads are not required if only the neck is going to be treated.

Massage With EMS Pads

The EMS pads (4) are used to treat the muscles of the abdomen, legs, arms, back or buttocks. The neck massage device can but does not have to be put on.

1. Clean and dry the area of the body being treated.
2. Plug the connector of the electrode cable (2) into the socket (1) on the device.
3. Attach the EMS pads using the press studs to the poles (3) of the electrode cable. **Both electrodes must be used for the device to work.**
4. Peel the protective films off the EMS pads and stick them onto the area being treated (Picture D). Do not overlap the EMS pads! Keep the protective films!

Operating the Device

1. Slide the On/Off switch (6) to the **ON** position to activate the device.
2. To switch on, press the **ON** button (12). The device starts stimulation in the first mode (running through all modes).
3. Use the **MODE** button (11) to set the mode you want. The modes are shown on the display (5) as follows:
 - 1 flash: Run through all modes
 - 2 flashes: Acupuncture
 - 3 flashes: Pressure massage
 - 1 + 3 flash: Acupressure
 - 2 + 3 flash: Kneading massage
 - 1, 2, 3 flash: Tapping massage
4. Keep increasing the intensity to a level where the treatment still feels pleasant. To do this, press the **HIGH** button (9), or use the **LOW** button (10) to decrease the intensity. There are a total of 16 intensity levels.

The device automatically switches off after 15 minutes. To switch it off before this, press the **ON** button.

5. After the treatment, slide the On / Off switch to the OFF position.
6. Clean the device after each use (see the "Cleaning and Storage" chapter).

CLEANING AND STORAGE

Please Note!

- Before all cleaning activities, disconnect the electrode cable (2) from the device and take off the EMS pads (4).
 - Do not use any sharp or abrasive cleaning agents or cleaning pads to clean the device. These may damage the surfaces.
-
- Clean the used EMS pads (4) under running water and allow them to dry in the air. **Do not use soap!** Cover the pads with the protective films when they are completely dry.
 - Wipe down the metal plates (8) and if necessary also the remaining parts of the device with a damp cloth. Then wipe them with a dry cloth.
 - Store the device and accessories in a dry place that is protected from dirt and direct sunlight and cannot be accessed by children or animals.

TROUBLESHOOTING

If the device does not work properly, check whether you are able to rectify the problem yourself. If the problem cannot be solved with the steps below, contact the customer service department.

Do not attempt to repair an electrical device yourself!

The EMS pads (4) no longer stick.

The electrode pads are not clean.

- ▶ Clean the pads under running water.

The device does not work.

The metal plates (8) or EMS pads (4) are not placed on correctly.

- ▶ Correct the fit. The whole of the surface of the metal plates or EMS pads must be in contact with the skin. There must not be any hairs, clothing, jewellery etc. in between.

There is not enough moisture between the metal plates (8) and the surface of the skin.

- ▶ Wet the skin and the metal plates with water.

The On/Off switch (6) is in the OFF position.

- ▶ Slide the On/Off switch to the ON position.

The batteries are too weak or inserted incorrectly.

- ▶ Insert new batteries or insert the batteries the other way round. Pay attention to the correct polarity when inserting the batteries!

The stimulation is too weak.

There is not enough moisture between the metal plates (8) and the surface of the skin.

- ▶ Wet the skin and the metal plates with water.

The metal plates (8) or EMS pads (4) are dirty.

- ▶ Follow the instructions in the “Cleaning and Storage” chapter.

The batteries are too weak.

- ▶ Insert new batteries.

DISPOSAL

Dispose of the packaging material in an environmentally friendly manner so that it can be recycled.

Batteries should be removed before disposing of the device and disposed of separately from the device at suitable collection points.

This device is governed by the European Directive 2012/19/EU on waste electrical and electronic equipment (WEEE). Do not dispose of this device as normal domestic waste, but rather in an environmentally friendly manner via an officially approved waste disposal company.

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