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Dear Customer,

We are delighted that you have chosen this **back support belt**. If you have any questions, contact the customer service department via our website:

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### Explanation of the Symbols and Signal Words

Danger symbol: This symbol indicates dangers of injury. Read through the associated safety notices carefully and follow them.

### Intended Use

- This product is intended to be used to provide temporary support and relief for the rib, abdominal and back muscles. In addition, it is intended for temporary posture support.
- The product is no substitute for necessary medical treatment.
- The product is designed for private use and not for commercial use.
- Use the product only for the intended purpose and as described in the operating instructions. Any other use is deemed to be improper.

### IMPORTANT NOTICES

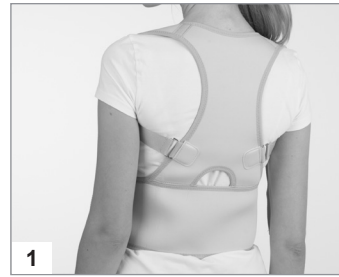
- Keep the packaging material away from children and animals. There is a danger of suffocation!
- The product is only intended for short-term support for the back, and not for therapeutic medical purposes. Before using the product and in the case of ongoing problems, you should consult a doctor.
- In the following cases, the product should **not** be used, **or only after consulting a doctor**:
  - slipped disc,
  - disc protrusion,
  - degenerative joint diseases,
  - diseases of the spine,
  - fractures of the spine, shoulders or ribs,
  - osteoporosis or
  - severe cardiovascular or respiratory diseases.
- Pregnant women should not use the product or use it only after consultation with their doctor.
- If you have any concerns about your health, consult a doctor before using the product.
- Do not use the product while sleeping.
- Do not fit the product too tightly. Breathing must not be restricted and the product must not press against the chest.
- If pain or discomfort occurs during or after use, remove the product immediately and consult a doctor.
- Protect the product from naked flames, extreme temperatures, continuous wet conditions as well as sharp and pointed objects.
- Keep the product out of the reach of children and animals in a dry, clean place which is protected from direct sunlight when it is not in use.

### Versions

06869 – Size S/M

06870 – Size L/XL

### Putting On

1

2

Pass your arms through the shoulder girdle and put the product on like a vest.

Place the abdominal belt left and right around the abdomen and fasten it at the front with the hook and loop fastener strap. The abdominal belt should fit tightly without hindering the breathing.

3

Pull down the protruding end of the shoulder girdle to tighten the fit. Tighten until shoulders are pulled back a little.

4

To release the shoulder girdle, lift the metal buckles at the front and pull them upwards.

### Material

64 % polyester, 36 % cotton, foam: polychloroprene

### Care

Hand wash

Do not bleach

Do not dry in a tumble dryer

Do not iron

Do not dry-clean

Do not use fabric softener.

Wash separately or with the same colours.

### Disposal

Dispose of the packaging material in an environmentally friendly manner so that it can be recycled.

If the product is no longer going to be used, dispose of it in an environmentally friendly manner as well.

ID of operating instructions: Z 06869\_06870 M DS V3 1220 md

Customer Service / Importer:

DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Germany • ☎ +49 38851 314650 \*)

\*) Calls to German landlines are subject to your provider's charges.

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland • ☎ +49 38851 314650 \*)