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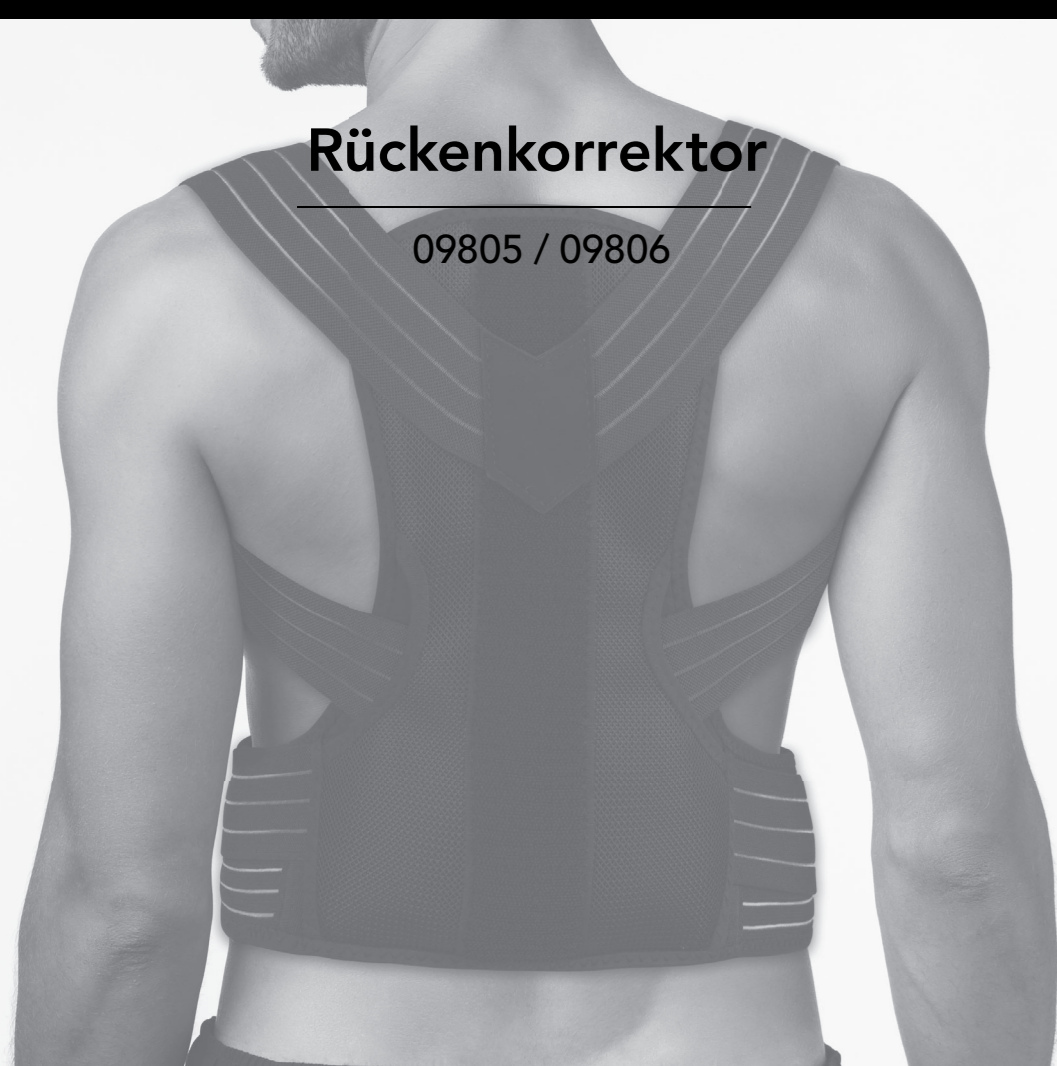
09805 / 09806

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EN Instructions for Use

Dear Customer, We are delighted that you have chosen the MAXXMEE posture corrector. Simply pull it on and adjust the straps with the hook-and-loop fastener to your back. One special extra: the gel pad which you can heat or cool. This can simply be inserted at the back and delivers a cosy warmth or a pleasant cooling feeling on the back.

If you have any questions about the product and about spare parts/accessories, contact the customer service department via our website: www.dspro.de/kundenservice

We hope you have a lot of fun with your MAXXMEE posture corrector.

Information About These Instructions

Before using the product for the first time, please read through these instructions for use carefully and keep them for future reference and other users. They form an integral part of the product. The manufacturer and importer do not accept any liability if the information in these instructions is not complied with.

SYMBOLS

Danger symbols: These symbols indicate possible dangers. Read the associated safety notices carefully and follow them.

Supplementary information

40°C coloured wash

Do not bleach

Do not dry in a tumble dryer

Do not iron

Do not dry-clean

SIGNAL WORDS

Signal words indicate hazards if the associated notices are not followed.

WARNING – medium risk, may result in serious injury or death

NOTICE – may result in risk of damage to material

ITEMS SUPPLIED

- 1 x posture corrector
- 1 x gel pad
- 1 x instructions for use

Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the product but contact our customer service department. Remove any possible films, stickers or transport protection from the product.

INTENDED USE

- This product is suitable for briefly supporting the back, but not for therapeutic medical purposes. It is essential to consult a doctor before using the product and if you experience persistent problems.
- The gel pad can be used as a cooling pad or a thermal support.
- The gel pad may only be heated in the microwave. Never place it in the oven!
- The product is not a substitute for medically required treatment.
- The product is for personal use only and is not intended for commercial applications.
- Use the product only for the specified purpose and only as described in these instructions for use. Any other use is deemed to be improper.
- The warranty does not cover faults caused by incorrect handling, damage or attempted repairs. The same applies to normal wear and tear.

SAFETY NOTICES

WARNING – Danger of Injury

- Keep children and animals away from the packaging material. There is a danger of suffocation.
- The product may not be used or may only be used following consultation with a doctor in the case of:
 - slipped disc,
 - disc protrusion,
 - degenerative joint diseases,
 - diseases of the spine,
 - fractures of the spine, shoulders or ribs,
 - osteoporosis and
 - serious cardiovascular or respiratory diseases.
- Pregnant women should not use the product or only after consulting their doctor.
- If you have concerns about your health, consult a doctor BEFORE using the product.
- People who are insensitive to heat or the cold and other vulnerable people who are unable to react to overheating / undercooling may not use the gel pad.
- Do not use the gel pad either hot or cold directly on the skin.
- Do not use the gel pad on body parts with no feeling.
- Do not use the product while sleeping.
- If pain or discomfort occurs during or after use, remove the product immediately and consult a doctor.

NOTICE – Risk of Damage to Material and Property

- Protect the product from naked flames, persistent wet conditions, direct sunlight and sharp or pointed objects.
- Never stand on the gel pad.
- Store the product in a dry, clean place so that it is protected from direct sunlight and is out of the reach of children and animals when it is not in use.

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* Calls to German landlines are subject to your provider's charges.

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USE

Please Note!

- Do not pull the straps too tight so that you do not constrict blood flow. Wearing the posture corrector should not be an unpleasant experience.
- To achieve success, the posture corrector should be worn regularly.

Posture Corrector

- Place your arms through the straps. Make sure that the posture corrector is positioned centrally between your shoulder blades.
- Assume a good posture: Straighten your back and stick out your chest.
- Close the posture corrector in front of your stomach using the hook-and-loop fastener.
- Tighten the straps so that they pull the shoulders back slightly. Fix using the hook-and-loop fastener.

Gel Pad

Heat the gel pad in the microwave or cool it in the freezer compartment if you wish.
Heat – for max. 40 – 60 seconds at 800 – 1200 watts
Cold – for approx. 2 hours in the freezer compartment

i Please note that the sensation of cold and heat differ for individual people depending on body temperature and sensitivity to temperature!

The gel pad can be used individually or together with the posture corrector. To use it with the posture corrector, place the gel pad in the inside pocket at the back.

CLEANING AND STORAGE

Please Note!

- Do not use any sharp or abrasive cleaning agents to clean the product. They may damage the surfaces.

- Wash the posture corrector following the instructions on the washing label if necessary. Do not use fabric softener. Wash separately or with similar colours.
- Wipe down the gel pad with a damp cloth if necessary.
- Allow the product to dry completely before storing it away.
- Store the product in a cool and dry place which is out of the reach of children and animals.

PRODUCT DATA

Article numbers: 09805 (Size S / M)
09806 (Size L / XL)
Materials: 50% polyurethane, 40% polyester, 10% elastane
ID of instructions Z 09805_09806 M DS V1
for use: 0820 uh

DISPOSAL

 Dispose of the packaging material in an environmentally friendly manner so that it can be recycled. The product should also be disposed of in an environmentally friendly way.

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All rights reserved.